


GENOVA METALS

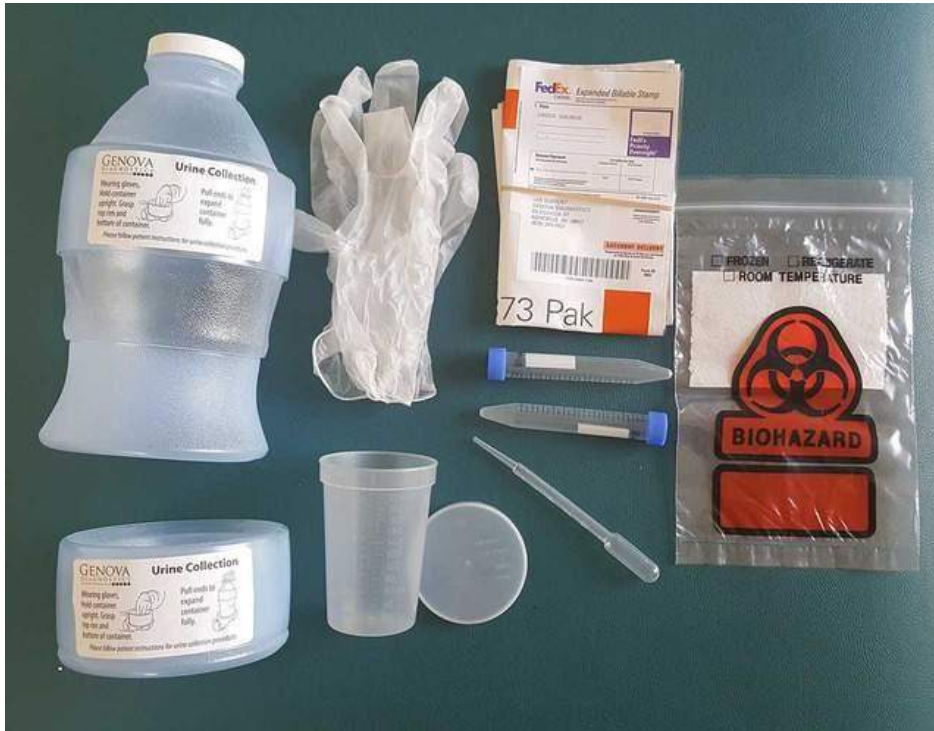

Please read the instructions thoroughly provided with this kit. Check your kit and call our office or the lab if anything is missing or damaged.

- Collect all urinary output within a 24-hour period.
- Skip the first urination of the day on Day 1. Note time of first collection. Collect all subsequent urinations into the collection cup for 24 hours, pouring urine into storage bottle after each urination.
- Place storage bottle in refrigeration in between urinations throughout the 24-hour collection period.
- Do not rinse or wipe collection cup. Simply place the lid on cup in between urinations.
- Collect and include the first urination on Day 2.
- If both storage bottles are used, add up the total volume of both bottles at end of the 24-hour period. Note total volume on the vials and on the requisition form.
- Place both vials into bio-hazard bag and place in box with the requisition on top.
- Take box to a Fed-Ex drop off as soon as possible.

Ensure that the requisition and all accompanying paperwork are filled out and placed in the package before sealing.

